


# OSTEOPLUG®

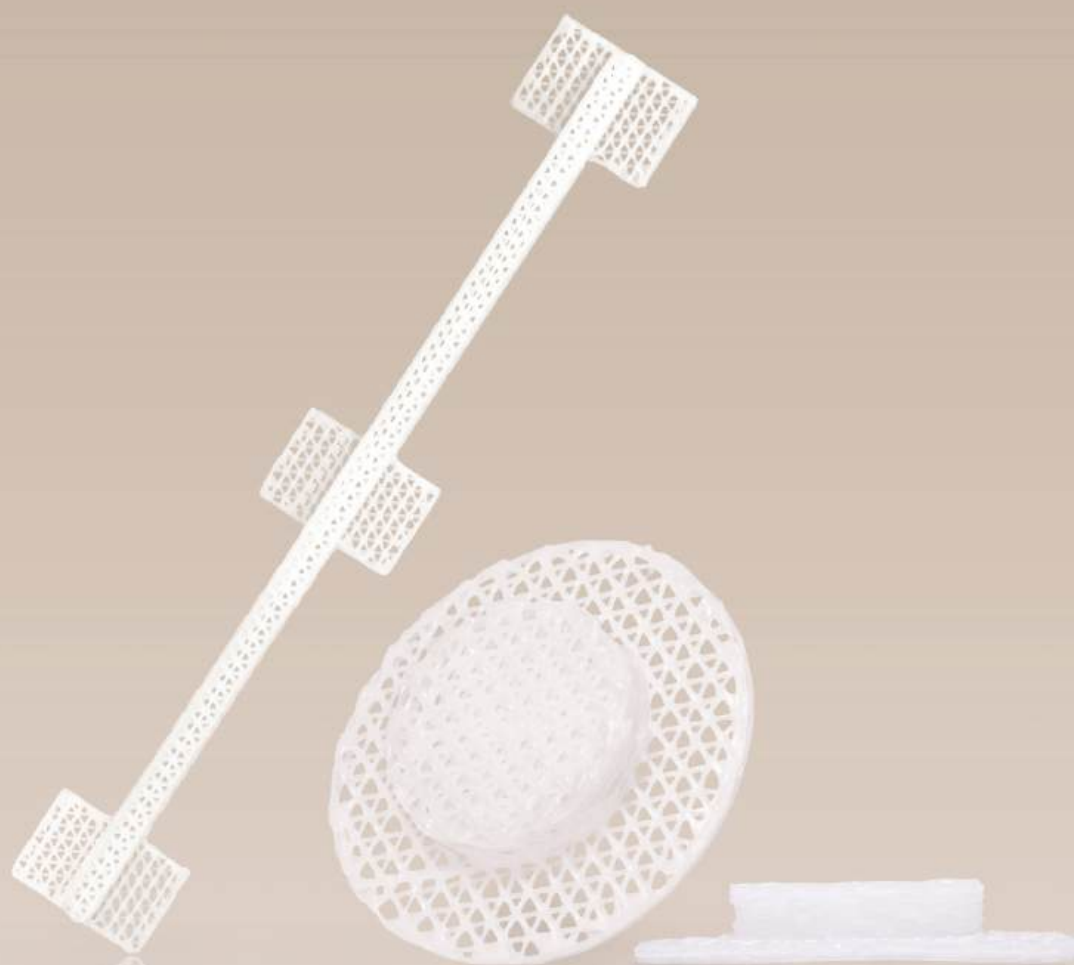

# 1

## BIOMIMETIC

- The **Osteoplug®** is a bioresorbable implant with a patented interconnected porous architecture that mimics the natural cancellous bone microstructure. It promotes tissue and vascular ingrowth.
- **Osteoplug®** is an integrating implant to restore burr hole defects and fresh craniotomy cuts by promoting tissue ingrowth, leading to a shift from purely cosmesis to functional regeneration of damaged tissues.
- **Osteoplug®** bears the CE mark of compliance, is FDA 510(k) cleared, fabricated in compliance with current Good Manufacturing Practice (cGMP) and EN ISO 13485 and provided sterile (gamma irradiation, EN ISO 11137).

# 2

## DESIGN

### 1. RESORBABILITY


- Polycaprolactone (PCL) is a biodegradable polymer that degrades and resorbs fully in vivo by hydrolysis which is then metabolized by the body.
- **Osteoplug®** has a gradual resorption profile, depending on the patient anatomy and metabolism, of approximately 18-24 months.
- **Osteoplug®** possesses optimal resorption rate that maintains mechanical integrity during healing process – minimizing adverse host-implant and inflammatory reactions.

### 2. POROSITY

- **Osteoplug®** is manufactured with a porous interconnected micro-architecture that demonstrates mechanical properties similar to human cancellous bone.
- Upon implantation, blood and surrounding cells are absorbed into the pores of the scaffold via capillary action – Creating a regenerative niche that is ideal for tissue formation.

### 3. SNAP-FIT DESIGN

- **Osteoplug®** is designed to fit perfectly into the burr hole defect and fresh craniotomy cuts. This ensures direct interaction between surrounding calvarial bone, allowing repair cells to use the implant as a scaffold, promoting bone tissue regeneration.

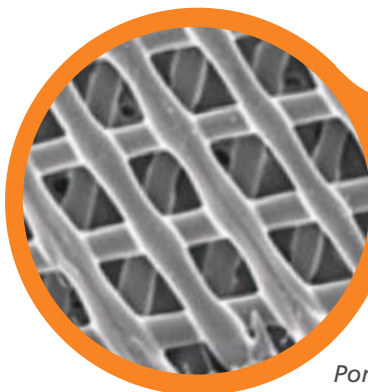

*Porosity of Osteoplug®*




## 3

## CLINICAL ADVANTAGE

## PATIENT'S PERSPECTIVE

- No patient developing adverse reactions such as pain, scarring, infections or excessive debris production.
- This implant has shown excellent functional and aesthetic outcomes.

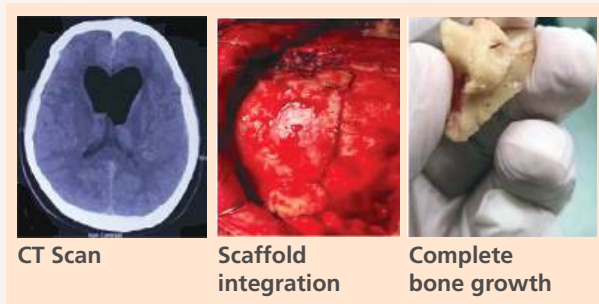

CT Scan

Scaffold  
integrationComplete  
bone growth

(Left) Patient's post-operative CT scan after 2 years, (Middle) Osteointegration seen between the scaffold and the rest of the calvarial bone, (Right) Physical appearance of the regenerated bone (previously Osteoplug®) after 2 years.

Data on file

## CLINICAL PERSPECTIVE

- Implanted since 2004 with no complications when used according to its approved Indications.
- Beyond 2 years of follow up shows host-implant compatibility with no infection and migration of implant.
- Avoided donor site morbidity.
- Designed to perfectly fit into burr holes formed by perforators and other cranial defects, allowing easy handling by surgeons and ensured excellent cosmesis.
- **Osteoplug®** were well-integrated into adjacent calvarial bone

## 4

## INDICATIONS FOR USE

**Osteoplug®** is indicated for the repair of neurosurgical burr holes and fresh craniotomy cuts.

## 5

## SURGICAL PROTOCOL

## 1. SITE PREPARATION

Prepare the implantation site using standard surgical techniques. Control of active bleeding should be achieved prior to implantation.

## 2. IMPLANT SELECTION

Select appropriate plug size that best suits burr hole defect.

## 3. IMPLANT PREPARATION

**Osteoplug®** is soaked with patient's blood.

## 4. INSERTION

Gently lower the **Osteoplug®** into the surgically created defect. The **Osteoplug®** would have a snug fit in the defect site. No fixation devices or suturing is required.

## 5. SCALP CLOSURE

Close the scalp using standard surgical techniques.


## 6

## HANDLING ADVANTAGE

- **Osteoplug®** conforms to the defect, thus maximizing direct contact with viable host bone.
- **Osteoplug®** is designed to perfectly fit into burr hole defects and craniotomy cuts, allowing easy handling by surgeons and ensured excellent cosmesis.
- **Osteoplug®** does not require fixation.

## 7

## OSTEOPLUG® SIZE CHART

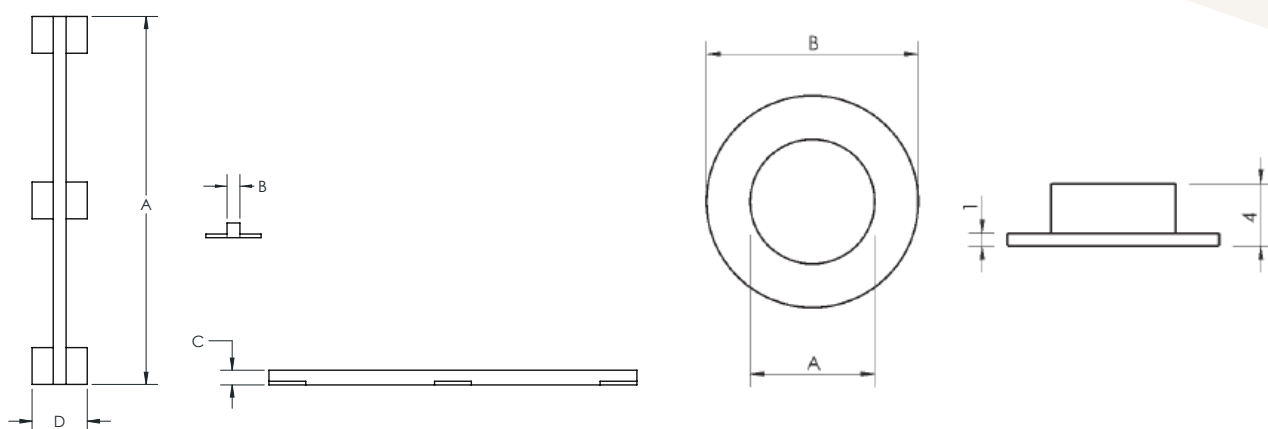

| PRODUCT CODE       | A   | B   | C | D  |
|--------------------|-----|-----|---|----|
| PC17 (100, 2, 4)   | 100 | 2   | 4 | 15 |
| PC17 (100, 3.5, 4) | 100 | 3.5 | 4 | 15 |

| Product Code    | A (mm) | B (mm) |
|-----------------|--------|--------|
| PC22(8, 8, 4)   | 8      | 14     |
| PC22(9, 9, 4)   | 9      | 15     |
| PC22(10, 10, 4) | 10     | 17     |
| PC22(11, 11, 4) | 11     | 19     |
| PC22(12, 12, 4) | 12     | 20     |
| PC22(13, 13, 4) | 13     | 22     |
| PC22(14, 14, 4) | 14     | 24     |
| PC22(15, 15, 4) | 15     | 25     |